

**RAYCAST® HIGH PRECISION BASE PLATE-TO-COUCH TOP FIXATION DEVICES LOXON™ FOR HP SYSTEM ON EXACT® COUCH TOPS**



Article No. : **32001**

**A. GENERAL PRODUCT INFORMATION**

The products referred to in these instructions are devices used to connect Raycast® HP base plates to an EXACT® couch top.

**B. PRODUCT DESCRIPTION**

LOXON™ for HP is designed for reproducible positioning and stable fixation of Orfit Industries base plates 32110, 32110/12 (HP Base Plate in CFL) and 35751N (HP Base Plate in HPL) on any couch with EXACT® indexing capabilities.


**C. PARTS LIST**

Article number	Description	pieces
32001/1	Set of the fixation devices	1
32001/3	Screw M6 x 12	2
32001/4	Spare O-ring, 20 x 2.5	2

**D. FIXATION TO BASE PLATES**

**Fixation to the 32110 and 35751N base plates**

LOXON™ consists of two separate pieces that are mounted on both sides of the base plate as shown above: one fixed piece and one moving piece with an eccentrically positioned cylinder on a lever.

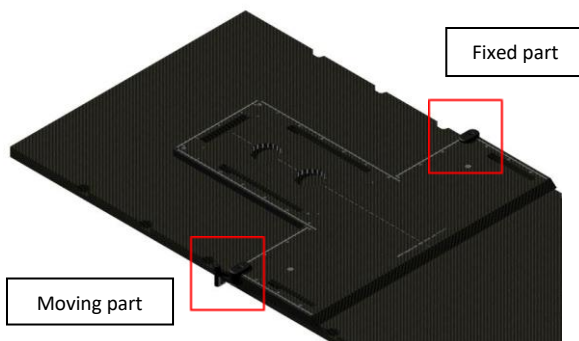

LOXON™ moving part



LOXON™ fixed part

The two parts of LOXON™ are mounted on the Orfit Industries base plate with the M6 x 12 screws supplied in the package. Always mount the fixation devices on the base plate before installing the base plate on the couch.

The lever of the moving part is rotated to fix the cylinder in the free space of the EXACT® indexing system.

#### E. STORAGE

Always store the product in a safe place to prevent it from getting damaged.  
Store the product between +10 °C (50 °F) and 40 °C (122 °F).

#### F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use.  
**Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.  
The product can be disposed of with household waste.

#### G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

#### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.  
RAYCAST® is a registered trademark of ORFIT INDUSTRIES N.V. LOXON™ is a trademark of Orfit Industries.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
welcome@orfit.com

[www.orfit.com](http://www.orfit.com)



Ref. No. 31803  
VERSION 6  
LAST UPDATE: 31/03/2021  
REVISION DATE: 31/03/2023