

HIGH PRECISION BASE PLATE-TO-COUCH TOP FIXATION DEVICES

LOXON™ FOR PELVICAST™ SYSTEM ON EXACT® COUCH TOPS



Article No. : **32002**

A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are devices used to connect PELVICAST base plates to an EXACT couch top.

B. PRODUCT DESCRIPTION

LOXON for PELVICAST is designed for reproducible positioning and stable fixation of Orfit Industries base plates 32122 (PELVICAST Base Plate in CFL), 32123 (PELVICAST Base Plate in HPL) and 35781-N1 (PELVICAST Base Plate in HPL, 1st generation) on any couch with EXACT indexing capabilities.

C. PARTS LIST


Article number	Description	Pieces
32002/1	Set of the fixation devices	1
32001/3	Screw M6 x 12	2
32001/4	Spare O-ring, 20 x 2.5	2

D. FIXATION TO BASE PLATES

Fixation to the 32110 and 35751N base plates

LOXON™ consists of two separate pieces that are mounted on both sides of the base plate as shown above: one fixed piece and one moving piece with an eccentrically positioned cylinder on a lever.

Fixed part

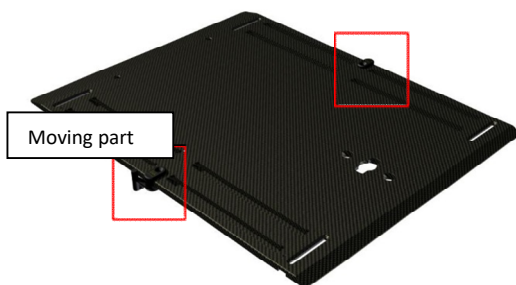

LOXON moving part



LOXON fixed part

The two parts of LOXON are mounted on the Orfit Industries base plate with the M6 x 12 screws supplied in the package.

Always mount the fixation devices on the base plate before installing the base plate on the couch.

The lever of the moving part is rotated to fix the cylinder in the free space of the EXACT indexing system.

LOXON for PELVICAST can be installed in three different pair of screw holes on the above mentioned base plates. It is also possible to attach two pairs of this fixation device to a base plate for extra stability.

E. PRECAUTIONS

Periodic checks of the products should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself**, but contact your distributor if there are any questions or concerns.

F. STORAGE

Always store the product in a safe place to prevent it from getting damaged.

G. MAINTENANCE AND WASTE MANAGEMENT

These products can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The products can be disposed of with household waste.

H. FIXATION DEVICES COMBINATION PACKAGES

This product is part of following fixation devices combination packages:

- 32155

I. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.
LOXON™ and PELVICAST™ are trademarks of Orfit Industries.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 31804
VERSION 5
LAST UPDATE: 21/03/2023
REVISION DATE: 21/03/2025