

THE AIO SOLUTION® SIDE PANNELS FOR INDIVIDUAL CUSHIONS



AIO SOLUTION® code: **AC-SP-L**
Art. No. : **32030** (4 pcs.)

A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is an accessory used for patient positioning and immobilisation in radiation therapy.

This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorized third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION

The side panels are a tool to help keep the base plate HP slots free while moulding the vacuum cushion so that a HP mask can still easily be made afterwards.

C. PARTS INFORMATION

The product is only compatible with the AIO base plates (Art. No. 32301, 32204 or any versions thereof) and Pelvicast base plates (Art. No. 32122, 32124 or 32123).

Parts you need		
	Art. No.	#
Side panels large set	32030	1 set
AIO or Pelvicast type base plate	32301, 32204, 32122, 32124, 32123 or any versions thereof	1 pcs.
Vacuum bags	18056, 18057 or others	1 pcs.

D. HOW TO USE

Insert the 4 side panels in the pelvic slots of the base plate. Secure the side panels with the EVA blocks. Position the vacuum cushion and patient on the base plate in between the side panels. Extract the air from the cushions by means of a vacuum pump. Remove the side panels (first the EVA blocks and then the side panels). The slots can now be used to fix the masks.

The side panels must be removed prior to radiation treatment !

E. STORAGE

Always store the side panels in a safe place to prevent them from getting damaged.
Store the side panels in between +10°C (50°F) and 40°C (122°F).

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

The instructions were written in accordance with the European Directive 93/42/EEC for Medical Devices. It is prohibited to make alterations to this text without prior approval from Orfit Industries.
THE AIO SOLUTION® is a registered trademark of ORFIT INDUSTRIES N.V. PELVICAST™ is a trademark of Orfit Industries.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 31869
VERSION 7
LAST UPDATE: 12/02/2019
REVISION DATE: 12/02/2021