
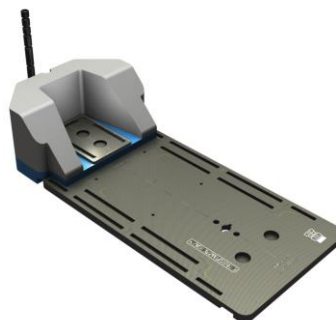


# THE AIO SOLUTION® AIO ARM REST ADAPTER FOR USE WITH VACUUM BAGS

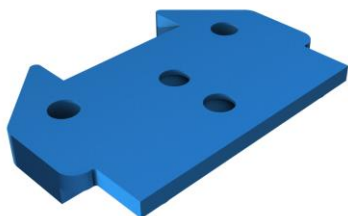

Article No. : 29036

## A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are medical devices, used for patient positioning and immobilisation in radiation therapy. This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

## B. PRODUCT DESCRIPTION

This AIO ARM REST ADAPTER is used to position the AIO arm rests when using the Orfit vacuum bags on the AIO base plate. Together with these products and EFFICAST® pre-cuts, it is used to form a reproducible patient positioning and immobilization system. Information on these other parts can be found in the respective 'instructions for use' and on [www.orfit.com](http://www.orfit.com).

## C. PRECAUTIONS FOR USE

Always use this cushion in combination with the other AIO SOLUTION® BREAST- & LUNGBOARD CUSHIONS on one of the base plates.

The cushion has indexing knobs on the bottom that fit into the positioning holes in the base plates. Always verify that the cushion is positioned correctly on the base plate.

Do not expose this cushion to a hot air blower and make sure that no sharp objects can come into contact with this cushion.

## D. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Do not put heavy objects on the cushion and prevent hard objects from falling onto it, because these will cause permanent deformations. Store the cushions between +10°C (50°F) and 40°C (122°F).

## E. PROPERTIES

### E.1. Basic Physical Properties

The following physical properties apply to these cushions:

Density: Foam: 29 kg/m<sup>3</sup>, Coating: > 1 g/cm<sup>3</sup>  
 Dimensions: L 380 mm x W 544 mm x H 210 mm  
 Weight: 1,30 kg

### E.2. Mechanical Properties

The cushion is made of low density PE foam that is covered with a PU coating. This combination results in a stable cushion that keeps its dimensions overtime and under conditions of frequent use.

The cushions used on the treatment machines are often used more frequently than those on the simulators and during the

imaging stages. We therefore recommend rotating the sets between the different machines on a regular base to avoid differences in wear and tear of the cushions.

The maximum deformation measured after 100 uninterrupted pressure cycles with a compression strength of 500N on a surface of 5cm x 5cm is 28.1%. This corresponds to the force of 100 times per day the load of 50 kg applied with the palm of the hand.

### E.3. Dosimetric Properties

The dosimetric build-up of this material (expressed in equivalent mmH<sub>2</sub>O) depends largely on the length of the radiation pathway inside the cushion. Each mm of foam passed by the radiation beam will add 0.02 mmH<sub>2</sub>O equiv. and each layer of coating that is passed will add another 0.35 mmH<sub>2</sub>O equiv. to the total. The total can thus be calculated using the following formula:

$$\text{mmH}_2\text{O} = (L \times 0.02) + (n \times 0.35), \text{ where:}$$

L = length of the radiation pathway in the foam (in mm)  
 n = number of coating layers crossed by the radiation beam

Attenuation (± 0.15%) per cm of material	
6 MV	15 MV
0.12 %	0.09 %

Note: Use these numbers as a guidance only. Perform the measurements again in your department to verify these results.

#### F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not**

**attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.  
The product can be disposed of with household waste.

#### G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

#### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.  
THE AIO SOLUTION® and EFFICAST® are registered trademarks of ORFIT Industries.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
[welcome@orfit.com](mailto:welcome@orfit.com)

[www.orfit.com](http://www.orfit.com)



Ref. No. 31999

VERSION 2

LAST UPDATE: 01/04/2021

REVISION DATE: 01/04/2023