


RAYCAST® HIGH PRECISION ACCESSORIES - PRONE HEAD SUPPORT

THE AIO SOLUTION® ACCESSORIES POSITIONING PLATE FOR PRONE HEAD SUPPORT

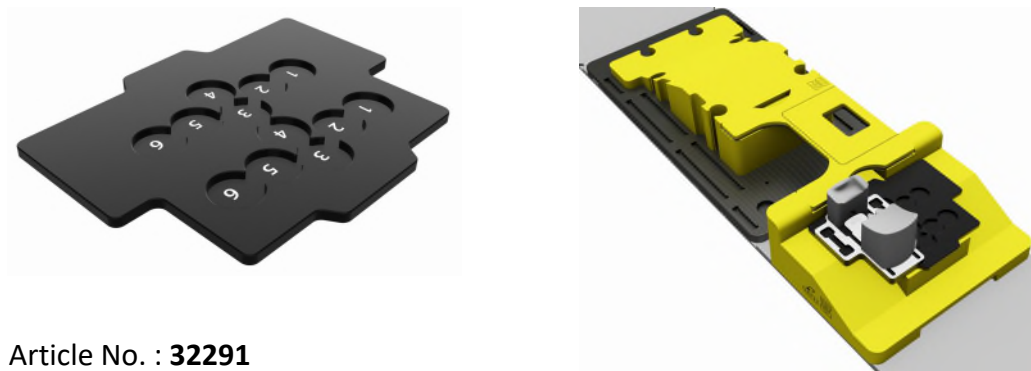

Article No. : **32291**

A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are medical devices, used for patient positioning and immobilisation in radiation therapy. This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION

This POSITIONING PLATE is an optional part of the "AIO SOLUTION® PRONE BREASTBOARD PACKAGE". It is used in combination with the other PRONE BREASTBOARD CUSHIONS, THE AIO SOLUTION® BASE PLATES and the EFFICAST® PRE-CUTS to form a reproducible patient positioning and immobilization system. It is used to position the PRONE HEAD SUPPORT (# 29050) laterally. Information on these other components of the AIO Solution® can be found in the respective 'instructions for use' and on www.orfit.com.

C. PRECAUTIONS FOR USE

Always use this POSITIONING PLATE in combination with the AIO SOLUTION® PRONE BREASTBOARD CUSHIONS on one of the AIO SOLUTION® base plates. The POSITIONING PLATE has two green knobs at the bottom to index it on the PRONE BREAST – CRANIAL CUSHION (# 29003).

The PRONE HEAD SUPPORT can be placed on the POSITIONING



PLATE in six different positions, depending on the position and size of the patient. The six positions are indicated by the numbers 1 to 6 on the plate.

D. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Store the product in such a way that it cannot fall or bump against other products. Never place the plate under load or pressure.

Store the device between + 10°C (50°F) and 40°C (122°F).

E. PROPERTIES

E.1. Basic Physical Properties

The following physical properties apply to this POSITIONING PLATE:

Dimensions: L 306 mm x H 19 mm x D 388 mm
Weight: 1,29 kg

E.2. Material Properties

The plate is made of PVC and the knobs are made of EVA foam.

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.
THE AIO SOLUTION® and EFFICAST® are registered trademarks of ORFIT Industries.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 50054
VERSION 3
LAST UPDATE: 05/10/2021
REVISION DATE: 05/10/2023