


RAYCAST® HIGH PRECISION BRAIN, HEAD & NECK INTERFACE FOR MAMMORX BREAST BOARD

Article No. : 39320

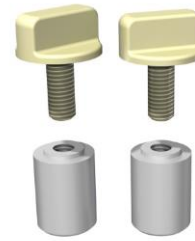

A. GENERAL PRODUCT INFORMATION

The product referred to in this document is an accessory for a medical device, used for patient positioning and immobilization in radiation therapy.

This product may only be used in combination with immobilisation masks produced by Orfit. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION

This interface allows to use 3-points and 5-points Head & Neck base plates on the MammoRx® tiltable breast board. With this solution, patients can be treated in a tiltable position with Orfit High Precision Masks (with 3, 4 and 5 points masks for Brain, Head & Neck). Information on these other parts (and instructions on how to make the masks) can be found in the respective 'instructions for use' and on www.orfit.com.


One interface set contains:

Article number	Description	Pieces
39320 / 1	Interface pins	2
39320 / 2	Wing screws	2

C. USAGE

To connect a base plate #32130 (Head), #32110 (Head & neck) & #32111 (Paediatric head & Neck) to the MammoRx, the #39320 interface set is used in combination with adapter plates:

- #35747/6 (adapter plate in metal – only the two lateral holes for handgrip poles can be used in this set up)
- #32063 (adapter plate in plastic, only three holes for handgrip poles can be used in this set-up: the two lateral holes and central hole are available)

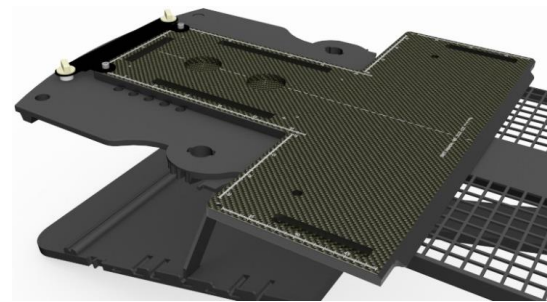

Important!

Using an adapter plate will disable the use of at least 2 holes for the hand grip poles:

- Aluminium adapter plate art.32063 covers holes 2, 3 & 4.
- Plastic adapter plate art. 32063 covers holes 2 & 3

When a Head and Neck Base Plate #32110 is used, the two lower positions for the arm supports will be covered and can't be used.

D. STORAGE

Always store the product in a safe place to prevent it from getting damaged or lost. Store the product between +10°C (50°F) and 40°C (122°F).

E. PROPERTIES

The interface pins are made of Acetal (POM) and the screws are made of nylon.

Pins: 27 mm x 19 mm x 19 mm

Screws: 32 mm x 25 mm x 25 mm

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.
RAYCAST® and EFFICAST® are registered trademarks of ORFIT Industries. PELVICAST™ is a trademark of Orfit Industries



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 50055
VERSION 2
LAST UPDATE: 02/04/2021
REVISION DATE: 02/04/2023