

SAGITTILT™ PRONE BREAST SOLUTION
SAGITTILT™ COMFORT HEAD REST



Article No. : **32070/16**

A. GENERAL PRODUCT INFORMATION

The product referred to in this document is a medical device, used in combination with Sagittilt Prone Breast Solution.



This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION

32070/16 is a head rest, which can be used as a comfortable alternative to the 'standard' forehead cushion of the Sagittilt Prone Breast Solution. The 32070/16 is a package and consists of the following items:

Information on these other parts (and instructions on how to make the masks) can be found in the respective 'instructions for use' and on www.orfit.com.



Qty.	Art. #	Description
1	32070/15	Sagittilt Interface for Comfort Head Rest 
1	29051	Comfort Head Rest 

C. USAGE

Place the tube of the interface in the head support-hole of the Sagittilt Prone Breast Solution as shown in the following image. The support can be placed in 2 height-positions ('1H' and '2H') using the indexing plunger underneath to switch positions (see red arrow in image).

D. STORAGE

Always store the product in a safe place to prevent it from getting damaged.

Store the product between +10°C (50°F) and 40°C (122°F).

E. PROPERTIES

32070/15
 Material: Aluminium & Galvanised steel, Velcro
 Dimensions: 176 x 250 x 235 mm
 Weight: 0,52kg

29051
 Material: Soft cushion in core, nonporous cloth skin.
 Dimensions: 75 x 290 x 290 mm
 Weight: 0,25 kg

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

The product 32070/15 can be disassembled and recycled. Item 29051 can be disposed with household waste.

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:
 It is prohibited to make alterations to this text without prior approval from Orfit Industries. SAGITTILT™ is a trademark of Orfit Industries



ORFIT INDUSTRIES
 Vosveld 9A | B-2110 Wijnegem | Belgium
 T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 50112
 VERSION 3
 LAST UPDATE: 02/04/2021
 REVISION DATE: 02/04/2023