



ORFIT Industries
 Vosveld 9a
 2110 Wijnegem, Belgium

BASE PLATE-TO-COUCH TOP FIXATION DEVICE FOR PRODIGY™ 2 TYPE COUCH

INSTRUCTIONS FOR USE

Article No. : **33167**



A. GENERAL PRODUCT INFORMATION

This product is a fixation device to attach the AIO, IMRT, SBRT and HP PRO long base plates to couch tops with the Prodigy™ 2 type indexing.

B. PRODUCT DESCRIPTION

This fixation device features 4 positioning blocks that can be attached on both sides of the long base plates of the AIO, IMRT, SBRT and HP PRO base plates to index on the Prodigy™ 2 style couches with a width of 53cm. This indexing device provides a highly reproducible patient positioning and immobilisation. Information on other parts, like the base plates, can be found in the respective 'Instructions for Use'.

Part list:

# of pieces	Ref. No.	Description
4	33167/1	Prodigy 2 Indexing Block
1	35744/3	Allen Wrench 4 mm
8	32066/2	M5 x 20 Screw

C. INSTRUCTION FOR USE

The 4 blocks of this fixation device have a circular recess that fits perfectly over the indexing knobs of the Prodigy™ 2 style indexing system.

Mount the indexing plates to the side of the base plate using the M5x20 screws and Allen wrench included in the package with the circular recess facing the couch edge.

Example: fixation on an IMRT Base Plate

A single set of 4 blocks guarantees the most stable setup. See below picture.



D. STORAGE

Always store the product in a safe place to prevent it from getting damaged.
 Store the product between +10°C (50°F) and 40°C (122°F).

E. PROPERTIES

E.1 Physical Properties

Dimensions of 1 block: W 36 mm x H 39 mm x L 9 mm
 Weight: 0.01 kg

The product is MRI compatible. It does not contain metallic elements except for the Allen wrench. Keep the Allen wrench out of an MRI room.

E.2 Density

Material POM (fixation plate): 1,4 gr/m³
 Material PEEK (screws): 1,32 gr/m³

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to make sure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

The instructions were written in accordance with the European Directive 93/42/EEC for Medical Devices. It is prohibited to make alterations to this text without prior approval from ORFIT Industries.

THE AIO SOLUTION® is a registered trademark of ORFIT Industries



Ref. No. 50120 - VERSION 2
 LAST UPDATE: 19/02/2016 - REVISION DATE: 19/02/2018