


## HP PRO<sup>®</sup> SOLUTION FLEXIBLE SHIM 1MM

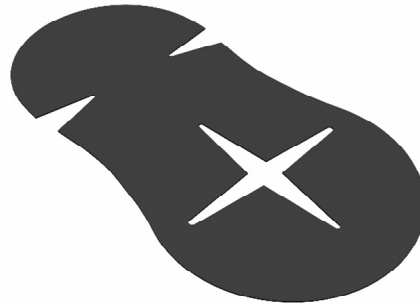

Article No. : 25070 /10

### A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is a medical device, used for patient positioning and immobilisation in radiation therapy.

This product may only be used in combination with immobilisation masks produced by Orfit. Orfit prohibits the use of unauthorized third-party products in conjunctions with its own products.

### B. PRODUCT DESCRIPTION

This Flexible Shim is an accessory of the ORFIT HP PRO SOLUTION. It is used in combination with the HP PRO base plate, a personalised head rest and special 3- and 5-point HP PRO masks to form a reproducible patient positioning and immobilisation device in the field of radiotherapy. Information on these other parts and instructions on how to make the masks can be found in the respective 'instructions for use' and on [www.orfit.com](http://www.orfit.com).

### C. USAGE

A flexible shim is used in combination with a personalised MOLDCARE CUSHION head rest. The shim is put on top of the head rest with the star opening facing caudally, as also mentioned on the shim.

This shim is used to counteract patient swelling and/or mask shrinkage.

If the shim is also being used during the treatment, it should be the same as the one used for the simulation. The same shim should remain with the patient throughout the whole treatment trajectory.



### D. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Store it in a dry place at a temperature of min. 10°C (50°F) and max. 30°C (86°F).

### E. PHYSICAL PROPERTIES

|            |  |
|------------|--|
| Dimensions | L 301 mm x W 156 mm x H 1 mm<br>L 11.85" x W 6.14" x H 0.04" |
| Weight     | 12 g<br>0.03 lbs   |
| Material   | EVA Hard-air foam  |
| WET value  | 0.4mm  |

### F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

The product can be disposed of with household waste.

### G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

#### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.  
HP PRO<sup>®</sup> is a registered trademark of Orfit Industries.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
[welcome@orfit.com](mailto:welcome@orfit.com)

[www.orfit.com](http://www.orfit.com)



Ref. No. 50143  
VERSION 3  
LAST UPDATE: 09/05/2023  
REVISION DATE: 09/05/2025