

HP PRO[®] SOLUTION PAEDIATRIC HEAD SUPPORT INDEXING PLATE



Article No. : **25000/39/20** (Package of 20 pieces)

A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is a medical device used for patient positioning and immobilisation in radiation therapy.

This product can be used during both the simulation and the treatment stage, including MRI simulation.

This product may only be used in combination with immobilisation masks produced by Orfit. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.


B. PRODUCT DESCRIPTION

The Head Support Indexing Plate is an accessory of the ORFIT HP PRO[®] SOLUTION. It is used in combination with the HP PRO[®] PEDIATRIC base plate 0-7 years old, special 3 and 5 point paediatric masks, a cranial stop and a personalised head rest to form a reproducible patient positioning and immobilisation device. Information on these other parts and instructions on how to make the masks can be found in the respective 'instructions for use' and on www.orfit.com.

C. USAGE

The Paediatric Head Support Indexing Plate is designed to attach a personalised head support, such as a Moldcare cushion, to any HP PRO[®] Paediatric base plate 0-7 years old. The indexing plate has two pins at the bottom that drop into two holes in the HP PRO[®] Paediatric base plates 0-7 years old. There are strips on the top surface that can be removed to expose the glue onto which the head support can be attached.

1. Remove the strips to expose the glue.

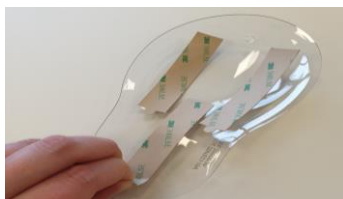

2. Attach the indexing plate to the base plate.



3. Attach the cushion to the indexing plate.



4. Position the patient on the Moldcare cushion and mould it to create enough support for the head and neck. Make sure to push the Moldcare cushion towards the paediatric indexing plate at the same time to prevent deformation of the indexing plate. Deformation might cause that the indexing plate pops out of the holes in the HP PRO[®] paediatric base plate 0-7 years old.

D. PHYSICAL PROPERTIES

Dimensions: L 180 mm x W 116 mm x H 18 mm
(curved sheet +/- 1 mm thick)
Weight : 0,018 kg (with tape)
Material: Polycarbonate (PC)

This product is made entirely of electrically non-conductive, non-metallic and non-magnetic materials and is MRI safe.'



E. STORAGE

Always store the product in a safe place to prevent it from getting damaged.

F. MAINTENANCE AND WASTE MANAGEMENT

The products can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.
HP PRO[®] is a registered trademark of Orfit Industries.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 50192
VERSION 2
LAST UPDATE: 13/04/2021
REVISION DATE: 13/04/2023