

HP PRO[®] SOLUTION PAEDIATRIC FLEXIBLE SHIM 1MM



Article No. : 25071/10

A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is a medical device, used for paediatric patient positioning and immobilisation in radiation therapy.

This product may only be used in combination with immobilisation masks produced by Orfit. Orfit prohibits the use of unauthorized third-party products in conjunctions with its own products.

B. PRODUCT DESCRIPTION

This Paediatric Flexible Shim is an accessory of the ORFIT HP PRO SOLUTION. It is used in combination with the paediatric HP PRO base plates, a personalised head rest and special 3- and 5-point paediatric HP PRO mask to form a reproducible patient positioning and immobilisation device in the field of radiotherapy. Information on these other parts and instructions on how to make the masks can be found in the respective 'instructions for use' and on www.orfit.com.

C. USAGE

A paediatric flexible shim is used in combination with a personalised MOLDCARE CUSHION head rest. The paediatric shim is put on top of the head rest with the V shaped opening facing caudally, as also mentioned on the paediatric shim. This paediatric shim is used to counteract patient swelling and/or mask shrinkage. If the shim is also being used during the treatment, it should be the same as the one used for the simulation. The same shim

should remain with the patient throughout the whole treatment trajectory.

D. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Store it in a dry place at a temperature of min. 10°C (50°F) and max. 40°C (104°F).

E. PHYSICAL PROPERTIES

Dimensions	L 206 mm x W 139 mm x H 1 mm L 8.11" x W 5.47" x H 0.04"
Weight	10 g 0.022 lbs
Material	EVA Hard-air foam
WET value	0.4mm

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.
HP PRO[®] is a registered trademark of Orfit Industries.

