

# HP PRO SOLUTION ELONGATED HAND HOLDS



Article No.: 25000/69/A & 25000/69/B

## A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are medical devices, used for patient positioning and immobilisation in radiation therapy. This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

## B. PRODUCT DESCRIPTION

The Elongated Hand Holds are accessories of the ORFIT HP PRO SOLUTION and ORFIT AIO 3.0 SOLUTION. It is used in combination with the HP PRO SOLUTION and AIO 3.0 SOLUTION products and masks to form a reproducible patient positioning and immobilisation device.

The Elongated Hand Holds are compatible with any HP PRO base plate and AIO 3.0 SBRT base plate (38013 & 38016).

Information on these other parts (and instructions on how to make the masks) can be found in the respective 'instructions for use' and on [www.orfit.com](http://www.orfit.com).

## C. USAGE

A hand hold can be attached to the left (B) and right (A) side of the base plates for the patient to hold onto.

### CAUTION

Make sure the handholds are applied after the patient has been positioned onto the base plate. This prevents it from being used as a handle for getting on the base plate, inducing forces for which it is not made. Failure in doing so may result in damage and/or injury to the patient.

There is a groove on the bottom on either side of the base plates and a rib on the handhold. Make sure the rib is positioned inside the groove before applying clamping force.



Turn the top portion of the handhold clockwise to clamp, see marking on top of the device. Do not use excessive clamping force, it is not required for a good fixation. The arrow shows at which position the handle is positioned on the base plate. There's an A and B side for asymmetrical positioning.



The Elongated Hand Holds can be used in overhang at the caudal end of the base plate.

## D. PHYSICAL PROPERTIES

Dimensions : L 335.7 mm x W 77 mm x H 35.5 mm  
Weight : 0,5 kg (1 piece)

## E. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Store the base plate between +10°C (50°F) and 40°C (105°F).

## F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

The product can be disposed of with household waste.

## G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com)

### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries. HP PRO® is a registered trademarks of ORFIT Industries.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
[welcome@orfit.com](mailto:welcome@orfit.com)

[www.orfit.com](http://www.orfit.com)



Ref. No. 50254  
VERSION 1  
LAST UPDATE: 09/01/2023  
REVISION DATE: 09/01/2025