

CLEANING INSTRUCTIONS

Orfit immobilization devices can be cleaned and disinfected by means of an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Do not soak.**



Never use aerosol sprays, corrosive cleaning agents, solvents, abrasive detergents or cleaning agents containing bleach.



OVERVIEW OF TESTED CLEANING AGENTS

The below cleaning agents have been tested on low density head supports and The AIO Solution 2.0 cushions (Low Density), regular density head supports (Regular Density), the AIO Solution 3.0 cushions (AIO 3.0) and the SBRT cushion.

Cleaning Agent	Low Density Head Supports and AIO 2nd Generation	Regular Density Head Supports	AIO 3.0	SBRT Cushion
AF Wipes	✗	✓	Not tested	Not tested
Alkalanet	✗	✗	Not tested	Not tested
Aniosurf ND Premium	✓	✓	✓	✓
Aniospray Quick	Not tested	✓	Not tested	Not tested
Bacillol 30 Tissues	✗	✓	Not tested	Not tested
Cidalkan wipes	✗	✗	✓	✗
Cidalkan spray	✗	✗	✓	✗
Clinell Detergent Wipes	✓	✓	✓	✓
Clinell Sporicidal Wipes	✓	✓	✓	✗
Clinell Universal Spray	✗	✓	✓	✗
Clinell Universal Wipes	✗	✓	✓	✗
Cleanisept wipes forte maxi	✓	✓	✓	✗
Clorox hydrogen peroxide disinfectant wipes	✗	✓	✓	✗
Descosept sensitive wipes	✓	✓	✗	✗
Diversey Optimax	✗	✓	✓	✗
Enzylin Actipur	✓	✓	✓	✗
Exeol Surf Optima	✓	✓	✓	✓



The
precision
expert

Cleaning Agent	Low Density Head Supports and AIO 2nd Generation	Regular Density Head Supports	AIO 3.0	SBRT Cushion
Franklab - Cidactiv	✗	✗	✗	✗
Franklab - FB wipes	✓	✗	✗	✗
Fugaten-spray	Not tested	✓	Not tested	Not tested
Hospisept-Tuch	Not tested	✓	Not tested	Not tested
Hydral 70	✓	✓	✓	✓
Incidin Alcohol Wipe	✓	Not tested	✓	✗
Incidin Foam	✗	✗	Not tested	Not tested
Incidin Liquid	✗	✗	✗	✗
Incidin Oxifoam	✗	✗	✗	✗
Incidin Oxyfoam S	✗	✗	✗	✗
Incidin Oxywipes	✗	✗	✗	✗
Incidin Oxywipes S	✗	✗	✗	✗
Incidin Plus	✓	✓	Not tested	Not tested
Mediwipes DM	Not tested	✓	Not tested	Not tested
Mediwipes Plus	Not tested	✓	Not tested	Not tested
Mikrozid AF Liquid	✗	Not tested	Not tested	Not tested
Orphi Ethanol 80%	Not tested	✓	Not tested	Not tested
Oxivir Excel Wipe CE	✗	✓	✓	✗
Oxivir TB	Not tested	✓	Not tested	Not tested
Pharma-C 70% Isopropyl Alcohol Wipes	✗	✓	✓	✗
Sani-Cloth AF3	✗	✓	✓	✗
Sani-Cloth Bleach	✗	✗	✗	✗
Sani-Cloth Plus	✗	✓	✓	✗
Sani-Cloth Super	✗	✓	✓	✗
Satpax - Presaturated Cleanroom wipers	✓	✓	✓	✗
Sensitive Wipes	✓	✓	Not tested	Not tested
Septalkan Spray - Alkapharm	✓	✓	✓	✓
Surfa Safe Premium	✗	✓	✓	✗
Surfanios Premium	✗	✗	✓	✗
Trionic	✓	✓	✓	✓
Velox Foam Extra	✓	✓	Not tested	Not tested
Virusolve +	✓	✓	✓	✓
Wecoline Clean 'n Easy Interior Cloths	✗	✓	Not tested	Not tested
Wip'Anios	✓	✓	✓	✗
70w% IPA (self-made solution)	✗	✓	✓	✗

Orfit is able to test cleaning agents that are not represented in this table.



For more detailed cleaning instructions,
[watch this video.](#)



www.orfit.com



ORFIT INDUSTRIES

Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

ORFIT INDUSTRIES AMERICA

810 Ford Drive | Norfolk, VA 23523
T 516-935-8500
welcome@orfit.com