



The  
precision  
expert

## CLEANING INSTRUCTIONS

Orfit immobilization devices can be cleaned and disinfected by means of an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Do not soak.**



Never use aerosol sprays, corrosive cleaning agents, solvents, abrasive detergents or cleaning agents containing bleach.



## OVERVIEW OF TESTED CLEANING AGENTS

The below cleaning agents have been tested on low density head supports and The AIO Solution 2.0 cushions (Low Density), regular density head supports (Regular Density), the AIO Solution 3.0 cushions (AIO 3.0) and the SBRT cushion.

| Cleaning Agent                              | Low Density Head Supports and AIO 2nd Generation | Regular Density Head Supports | AIO 3.0      | SBRT Cushion |
|---|--|-------------------------------|--------------|--------------|
| ACTO GmbH - Actoanid wipes                  | ✗  | ✓                             | ✓ (top side) | ✓            |
| AF Wipes                                    | ✗  | ✓                             | Not tested   | Not tested   |
| Alklanet                                    | ✗  | ✗                             | Not tested   | Not tested   |
| Aniosurf ND Premium                         | ✓  | ✓                             | ✓            | ✓            |
| Aniospray Quick                             | Not tested                                       | ✓                             | Not tested   | Not tested   |
| Bacillol 30 Tissues                         | ✗  | ✓                             | Not tested   | Not tested   |
| Cidalkan wipes                              | ✗  | ✗                             | ✓            | ✗            |
| Cidalkan spray                              | ✗  | ✗                             | ✓            | ✗            |
| Clinell Detergent Wipes                     | ✓  | ✓                             | ✓            | ✓            |
| Clinell Sporicidal Wipes                    | ✓  | ✓                             | ✓            | ✗            |
| Clinell Universal Spray                     | ✗  | ✓                             | ✓            | ✗            |
| Clinell Universal Wipes                     | ✗  | ✓                             | ✓            | ✗            |
| Cleanisept wipes forte maxi                 | ✓  | ✓                             | ✓            | ✗            |
| Clorox hydrogen peroxide disinfectant wipes | ✗  | ✓                             | ✓            | ✗            |
| Descosept sensitive wipes                   | ✓  | ✓                             | ✗            | ✗            |
| Diversey Optimax                            | ✗  | ✓                             | ✓            | ✗            |
| Enzylin Actipur                             | ✓  | ✓                             | ✓            | ✗            |
| Exeol Surf Optima                           | ✓  | ✓                             | ✓            | ✓            |



# The precision expert

| Cleaning Agent                         | Low Density Head Supports and AIO 2nd Generation | Regular Density Head Supports | AIO 3.0    | SBRT Cushion |
|--|--|-------------------------------|------------|--------------|
| Franklab - Cidactiv                    | ✗  | ✗                             | ✗          | ✗            |
| Franklab - FB wipes                    | ✓  | ✗                             | ✗          | ✗            |
| Fugaten-spray                          | Not tested                                       | ✓                             | Not tested | Not tested   |
| Hospisept-Tuch                         | Not tested                                       | ✓                             | Not tested | Not tested   |
| Hydral 70                              | ✓  | ✓                             | ✓          | ✓            |
| Incidin Alcohol Wipe                   | ✓  | Not tested                    | ✓          | ✗            |
| Incidin Foam                           | ✗  | ✗                             | Not tested | Not tested   |
| Incidin Liquid                         | ✗  | ✗                             | ✗          | ✗            |
| Incidin Oxifoam                        | ✗  | ✗                             | ✗          | ✗            |
| Incidin Oxyfoam S                      | ✗  | ✗                             | ✗          | ✗            |
| Incidin Oxywipes                       | ✗  | ✗                             | ✗          | ✗            |
| Incidin Oxywipes S                     | ✗  | ✗                             | ✗          | ✗            |
| Incidin Plus                           | ✓  | ✓                             | Not tested | Not tested   |
| Mediwipes DM                           | Not tested                                       | ✓                             | Not tested | Not tested   |
| Mediwipes Plus                         | Not tested                                       | ✓                             | Not tested | Not tested   |
| Mikrozid AF Liquid                     | ✗  | Not tested                    | Not tested | Not tested   |
| Orphi Ethanol 80%                      | Not tested                                       | ✓                             | Not tested | Not tested   |
| Oxivir Excel Wipe CE                   | ✗  | ✓                             | ✓          | ✗            |
| Oxivir TB                              | Not tested                                       | ✓                             | Not tested | Not tested   |
| Pharma-C 70% Isopropyl Alcohol Wipes   | ✗  | ✓                             | ✓          | ✗            |
| Sani-Cloth AF3                         | ✗  | ✓                             | ✓          | ✗            |
| Sani-Cloth Bleach                      | ✗  | ✗                             | ✗          | ✗            |
| Sani-Cloth Plus                        | ✗  | ✓                             | ✓          | ✗            |
| Sani-Cloth Super                       | ✗  | ✓                             | ✓          | ✗            |
| Satpax - Presaturated Cleanroom wipers | ✓  | ✓                             | ✓          | ✗            |
| Sensitive Wipes                        | ✓  | ✓                             | Not tested | Not tested   |
| Septalkan Spray - Alkapharm            | ✓  | ✓                             | ✓          | ✓            |
| Surfa Safe Premium                     | ✗  | ✓                             | ✓          | ✗            |
| Surfanios Premium                      | ✗  | ✗                             | ✓          | ✗            |
| Trionic                                | ✓  | ✓                             | ✓          | ✓            |
| Velox Foam Extra                       | ✓  | ✓                             | Not tested | Not tested   |
| Virusolve +                            | ✓  | ✓                             | ✓          | ✓            |
| Wecoline Clean 'n Easy Interior Cloths | ✗  | ✓                             | Not tested | Not tested   |
| Wip'Anios                              | ✓  | ✓                             | ✓          | ✗            |
| 70w% IPA (self-made solution)          | ✗  | ✓                             | ✓          | ✗            |

Orfit is able to test cleaning agents that are not represented in this table.



For more detailed cleaning instructions, [watch this video.](#)

