



ORFIZIP® LIGHT NS

orfit The personal fit

IN ORTHOTIC FABRICATION
FOR PHYSICAL REHABILITATION

Recommended applications

- Lightweight circumferential orthoses for immobilization and protection following trauma and/ or surgery.
- Foamy surface is especially beneficial for patients with sensitive and/ or fragile skin.
- Use with mobilization orthoses: outriggers are easily bonded to the surface and the base orthosis will not migrate distally.
- Offers a snug fit without straps.

Benefits

Highly conforming

Orfizip Light NS completely conforms to the anatomy of the hand, wrist, and forearm, providing exact contour and detail.

Time saving

No need to make a pattern, no straps and a quick fabrication method.

Increase patient adherence

Help your patients to adhere to their wearing schedule by treating them with a product that is easily accepted due to its lightweight material, design, comfort, and aesthetic appearance.

Effortless fabrication

Three features that make fabrication easier:

- ✓ The zipper aids in alignment while molding
- ✓ Non-perforated edges facilitate finishing of the orthosis
- ✓ The orthosis is completely remoldable when there is a change in patient's condition

Hygienic and easy to clean

A special coating prevents antibacterial growth and reduces unpleasant odors while wearing the orthosis. The orthosis is easy to deep clean in the washing machine.

Sizes

Orfizip Light NS Size	Wrist circumference in centimeters	Wrist circumference in inches
Small	12- 15	4.72 - 5.9
Medium	16 -19	6.29 - 7.48
Large	20 - 25	7.87 - 9.84

ORFIZIP® LIGHT NS

Activate, zip and mold!

Orfizip Light NS is a zipper closed pre-cut wrist immobilization orthosis. The double sided zipper offers the ability to fabricate either left or right handed orthoses. The shape, design, perforations and soft surface ensures a snug and comfortable fit around the wrist and forearm. Patients will appreciate the easy on-easy off zipper closure and therapists will be amazed at how easily and quickly the orthotic fabrication is completed.



ORFIZIP® LIGHT NS ATOMIC BLUE

Thickness (mm-inch)	Perforation types	Size	Art. N°
2.5 mm - 3/32"	micro	small	39108MB/25MI/L/NS
	perforated	medium	39109MB/25MI/L/NS
3.2 mm - 1/8"	micro	medium	39112MB/32MI/L/NS
	perforated	large	39113MB/32MI/L/NS

About ORFIT

Orfit Industries develops and produces thermoplastic materials that offer the highest possible performance to both therapists and patients in physical rehabilitation. Through workshops, inservices and online education, our experts provide tips and tricks to make your orthotic fabrications functional and successful.

Discover our complete family of low temperature thermoplastics that will help you solve your most challenging orthotic fabrication requirements on www.orfit.com

Discover step-by-step guides to specific orthosis fabrication in the Orfit Academy, www.orfit.com/academy.

Visit
WWW.ORFIT.COM
for more information.

www.orfit.com



ORFIT INDUSTRIES
Vosveld 9A
B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

ORFIT INDUSTRIES AMERICA
810 Ford Drive
Norfolk, VA 23523 | USA
T 516-935-8500
welcome@orfit.com